

ORLIMAN LINERS



Model indicated for patients with low to moderate activity.



Model indicated for highly active patients.

Ref.: LI1001 SOFT

Ref.: LI1002 ACTIVE

SILICONE TIBIAL LINER WITH PIN.

Characteristics:

Different densities are available depending on the patient's activity, thus providing the resistance and comfort demanded by amputees with different activities and characteristics.

Indications:

Indicated for patients with tibial amputations.

Properties:

- Manufactured in medical silicone.
- Matrix of 10 cm to stabilise the distal soft tissues.
- Distal umbrella designed to distribute the forces over the entire surface. Universal screwed orifice that allows it to be adapted to all kinds of suspension systems. (Shuttle locks).

Size selection:

Size selection is extremely important as an incorrect size could cause sweating and lack of precision when adapting the prosthesis and when walking; the stabilising matrix and distal umbrella should remain perfectly adapted to the stump. For this reason, Orliman has a broad range of sizes, from 20 up to 40 cm, to meet the requirements of different tibial stump morphologies.

For selection of the correct size measure on the patient's stump 4 cm proximally from the distal end. Then, select a size less than the measurement obtained and try it on the patient for them to check it before casting. A liner two sizes smaller is recommended for amputees undergoing their first prosthetisation in anticipation of fast stump shrinkage and possible liner displacement.

Colour:

- Blue.

SIZES CM	20	22	23,5	25	26,5	28
	30	32	34	36	38	40